

ChemoINTEL™: A high-throughput, multi-parameter compound screening platform to personalize cancer treatment by guiding therapeutic selection

Puneet Singh¹, Paulina Sindrewicz-Góral¹, Nupur Gupta¹, Seija Nevalainen¹, Santosh Putta², Norman Purvis¹

¹Pierian Biosciences Ltd, Liverpool, UK

²BioLegend Inc, San Diego, CA, USA

ABSTRACT

Background: Cancer is the leading cause of death worldwide 9.7 million deaths in 2022. There were 20 million new diagnosed cases of cancer in 2022 and an estimated increase to 35 million by 2050. Most patients undergo cytotoxic chemotherapy as the “backbone” of their cancer treatment driven by evidence-based guidelines. However, each patient tumour is unique and requires personalized therapeutic guidance. Pierian Biosciences has developed the ChemoINTEL™ platform to quantify patient tumour cell responses to chemotherapeutics and guide physicians in the selection of the most appropriate chemotherapy.

Methods: ChemoINTEL™ platform utilizes automated imaging of single tumour cells providing real-time kinetics of induced apoptosis and cell death to panel of treatment conditions and predicts patient response using internally developed algorithms. By combining different treatment conditions, the platform provides intelligent design of single agent or combination treatment approaches. As a compound library screening tool or potential clinical diagnostic, Pierian Biosciences’ quality-controlled processes ensure all its equipment, reagents and processes follow ISO17025 guidelines to ensure the quality of all data generated.

Results: Drug induced apoptosis have been measured for a wide range of tumour types where patient specimens were treated with chemotherapeutic agents as single drugs or in combination. Synergistic as well as antagonistic effect of combination therapy is also evaluated.

Conclusions: ChemoINTEL™ platform can be utilized for both, drug development and clinical diagnostic, predicting the most effective treatment regimen for cancer treatment.

INTRODUCTION AND OBJECTIVES

Predict chemotherapeutic treatments for cancer that are specific to the patient through an *in vitro* assessment of drug responses in enriched tumour cells


- The selection of cytotoxic drugs used for cancer treatment continues to be guideline driven and based on:
 - clinical evidence from treatment protocols developed from large, population-based, phase 3, prospective, randomized, multi-centre studies
 - patient cohort treatment outcomes (progression free survival [PFS] and overall survival [OS])
- This approach was necessitated by the stark reality that no “predictive” or “treatment-directing” diagnostic technologies were available, a circumstance that, to an overwhelming degree, remains unaltered at this time. However, every patient is unique, and their clinical response is varied.
- ChemoINTEL™ assay predicts tumour responses to cytotoxic treatment in individual patients and identifies which chemotherapeutic agent a patient is most likely to respond to.
- Identifying which specific drugs is going to be the most effective for a patient is needed and would lead to significant benefits including:
 - Improved patient outcomes to 1st Line Treatment
 - Reduced toxicity-associated side effects through decreased use of less effective agents
 - Reduced cost and improved quality of life
- The currently ongoing ChemoINTEL™ Algorithm Development Study aims to develop a predictive algorithm by investigating the correlation between *in vitro* cytotoxic drug-induced apoptosis and clinical response to administered chemotherapy in epithelial ovarian cancer patients.

METHODS AND WORKFLOW

The ChemoINTEL™ platform utilizes a semi-automated process for sample preparation, plating, and imaging


- Tumour specimens are shipped to Pierian Biosciences under temperature-controlled conditions for processing within 24 hours of surgical resection
- | Pre-Analytical Steps | Sample Processing | Analytical Steps |
|---|--|--|
| <ul style="list-style-type: none"> Sample collection and shipment Sample receipt and processing within 24 hours | <ul style="list-style-type: none"> Semi-automated dissociation of tissues into single-cell suspensions using benchtop gentleMACS™ Dissociator Automated enrichment of tumour cells using benchtop autoMACS™ ProSeparator Hamilton Liquid Handling robot (master drug plate preparation, sample plating and drug stamping) BioTek BioSpra 8 Incubator and BioTek Cytation 10 (for real-time kinetic imaging system) | <ul style="list-style-type: none"> Sample Acquisition: automated imaging and Flow Cytometry Automated gating |

Figure 1. Pierian Biosciences ChemoINTEL™ platform captures drug-induced cellular responses

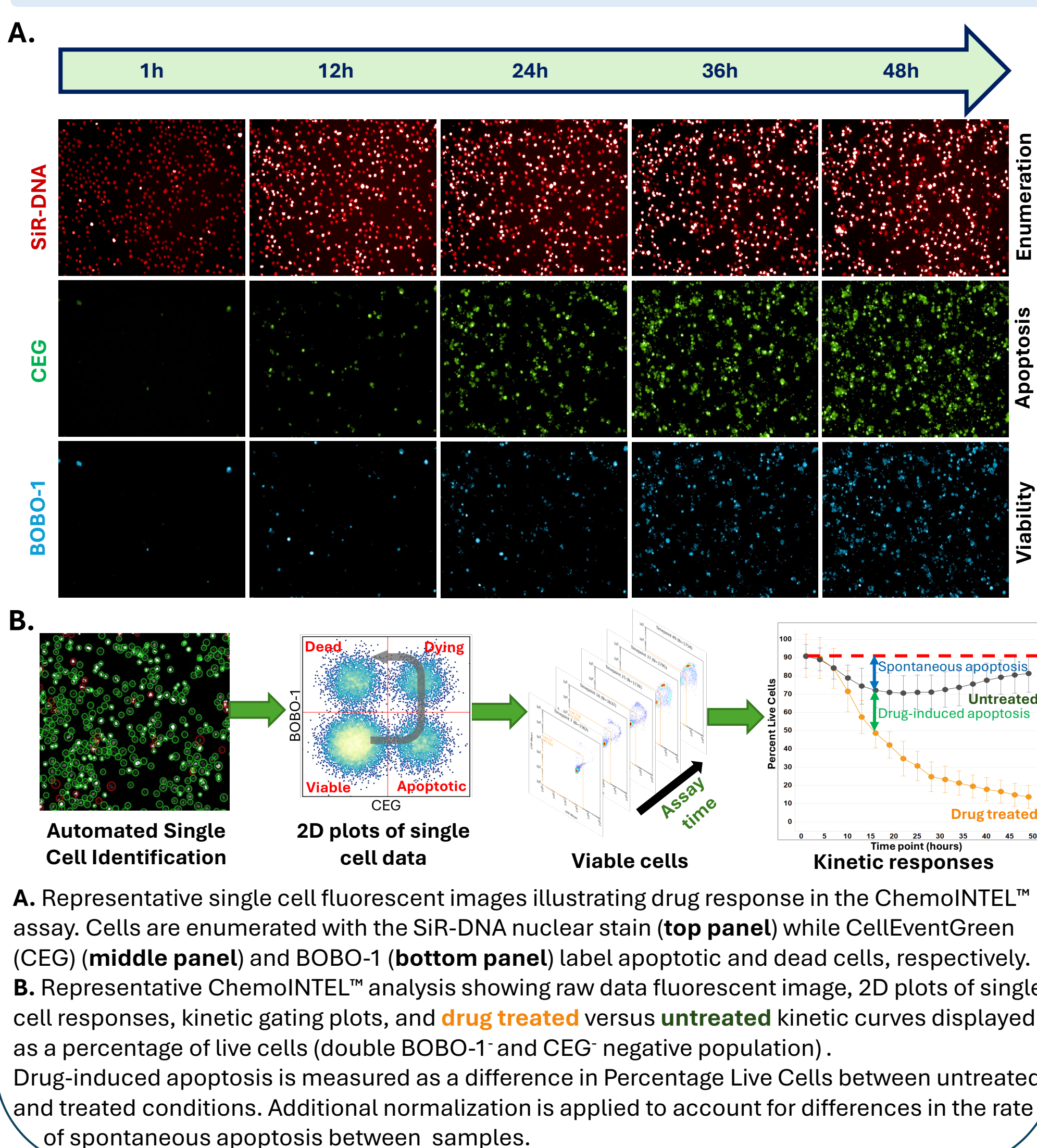

RESULTS

Figure 2. On-plate Cell Line Assay Quality Control

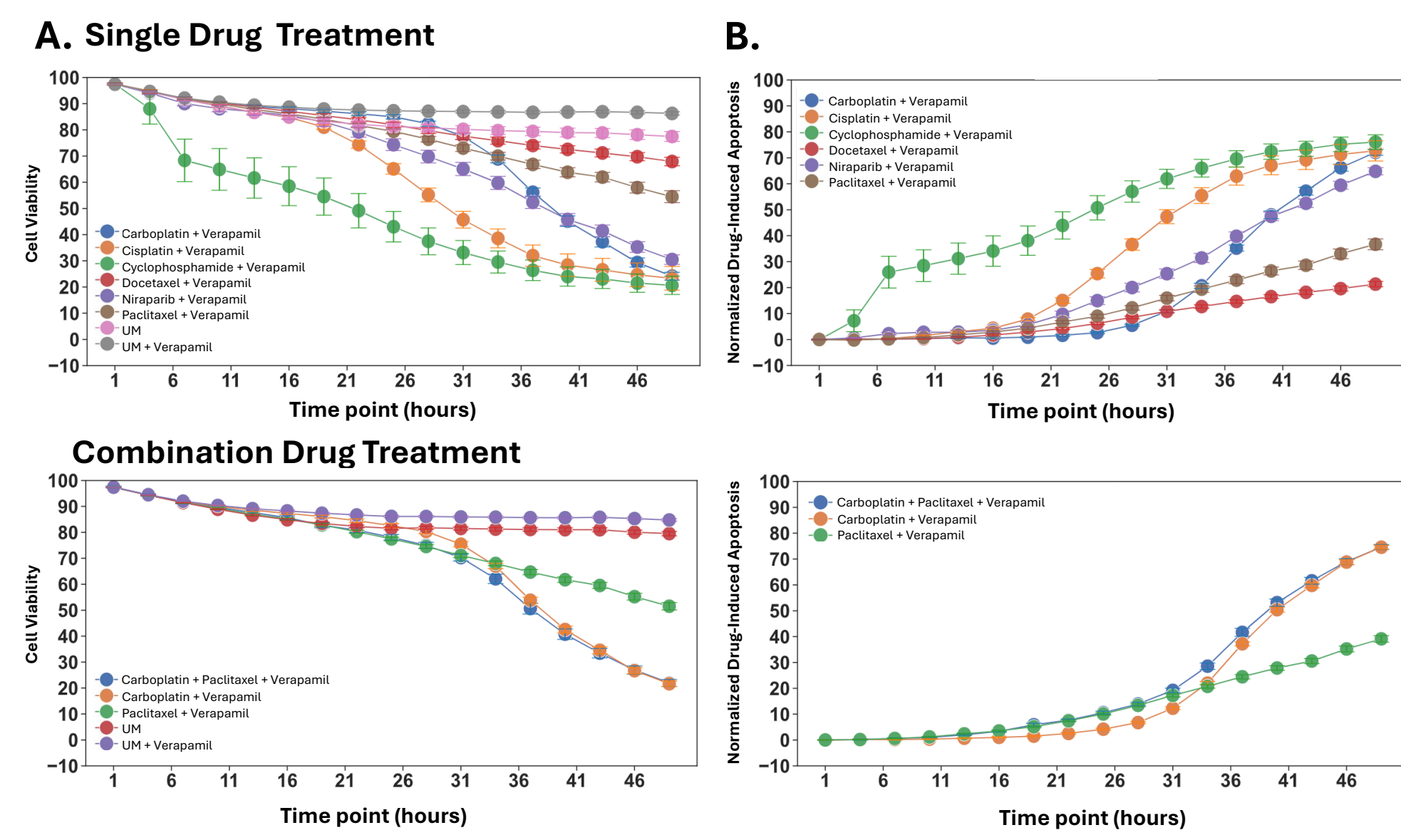

JURL-MK2 cells treated with chemotherapeutic agents and fluorescent probes for real-time imaging of cell viability and drug-induced apoptosis

- Carboplatin, Cisplatin, Cyclophosphamide, Docetaxel, Niraparib and Paclitaxel
 - Single drug treatments (top panels) and combination treatments (bottom panels)
- A.** The percentage of non-apoptotic (live) cells plotted as a function of time (left panel)
- B.** The amount of drug-induced apoptosis in treated cells calculated and plotted as a function of time (right panel)

Figure 3. ChemoINTEL™ assay performance monitoring and longitudinal tracking

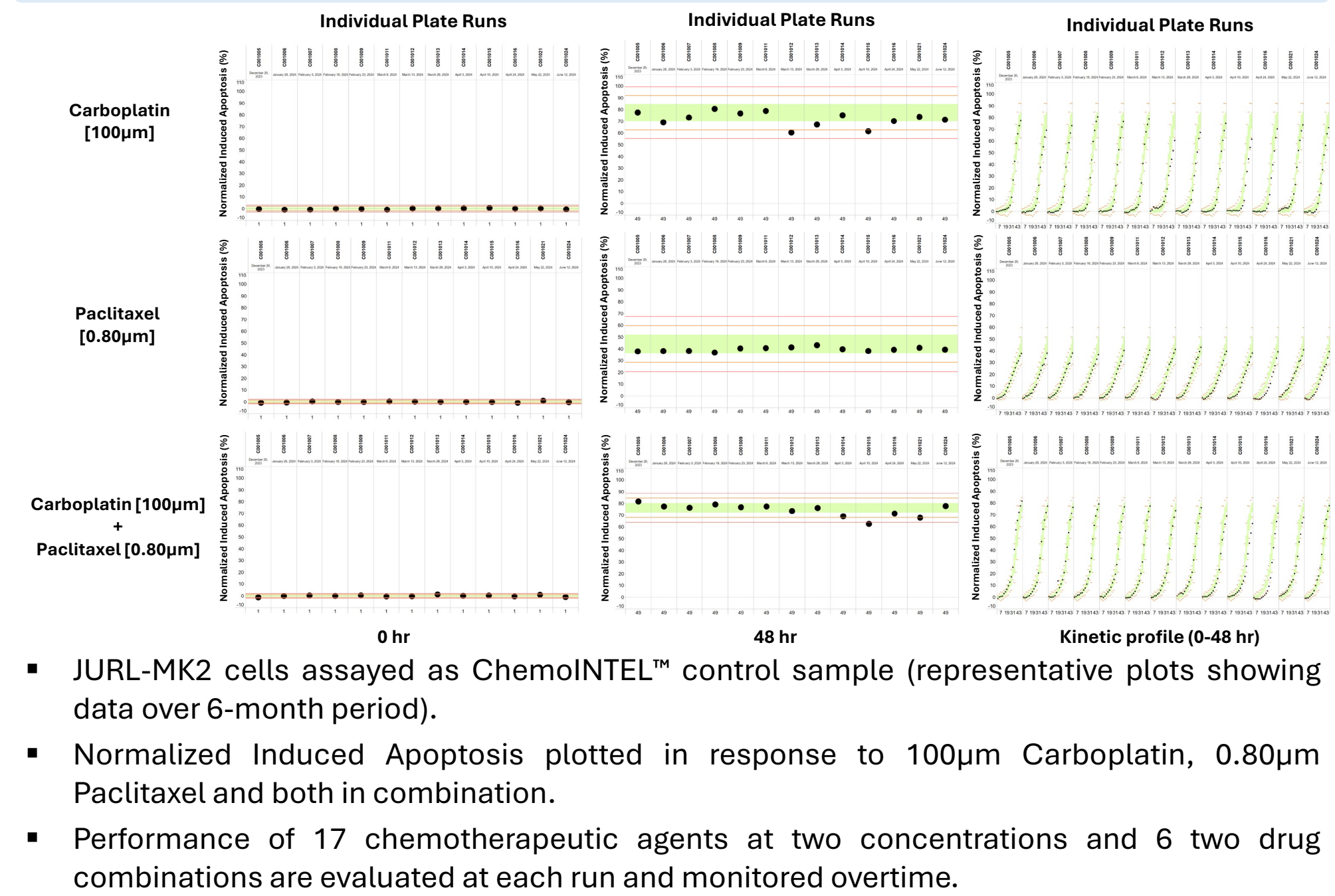


Figure 4. Pierian Biosciences ChemoINTEL™ technology can be applied to primary tumour processing and enrichment



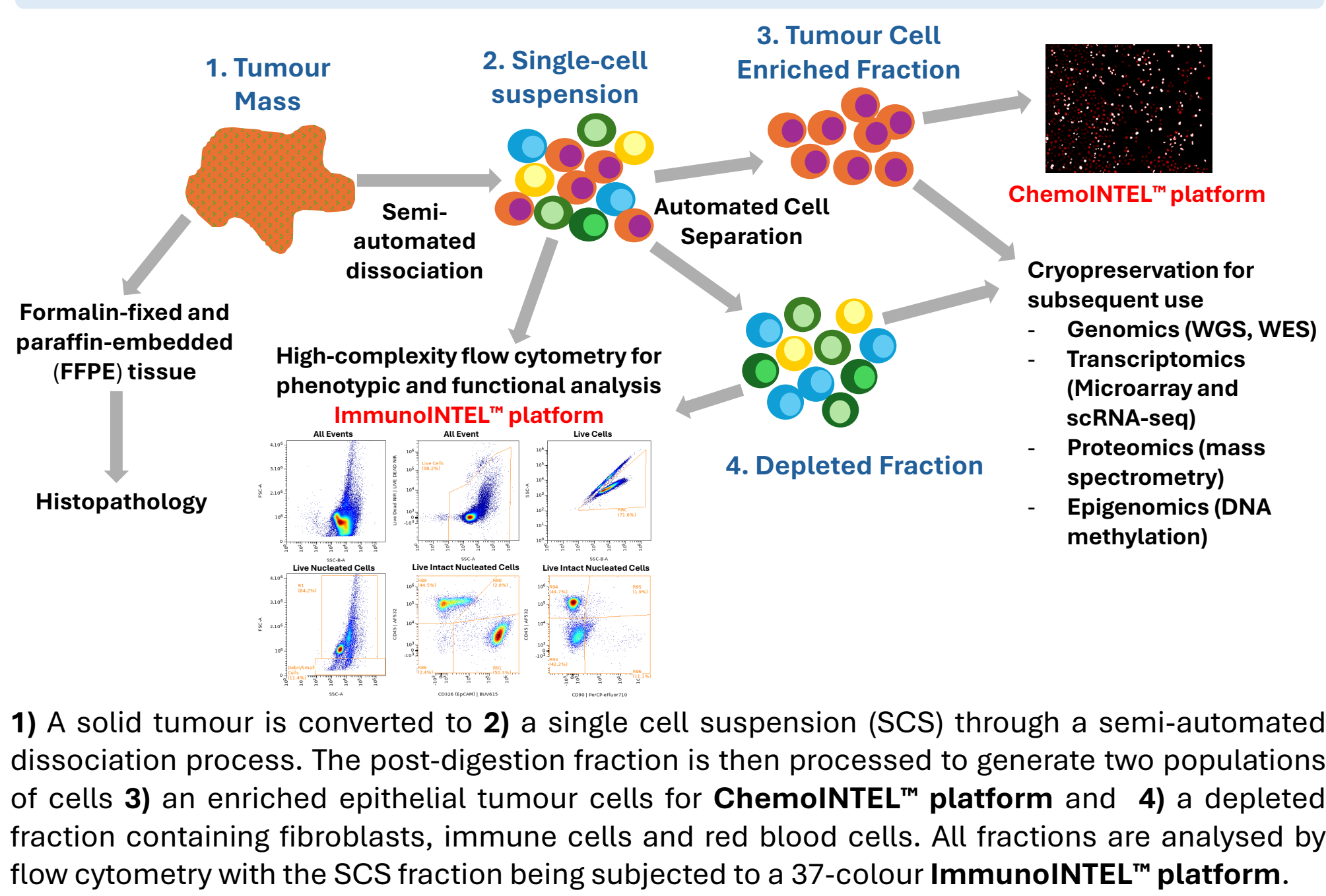


Figure 5. A wide range of tumour cell viabilities are observed over the course of the ChemoINTEL™ assay, establishing limits for statistically significant response

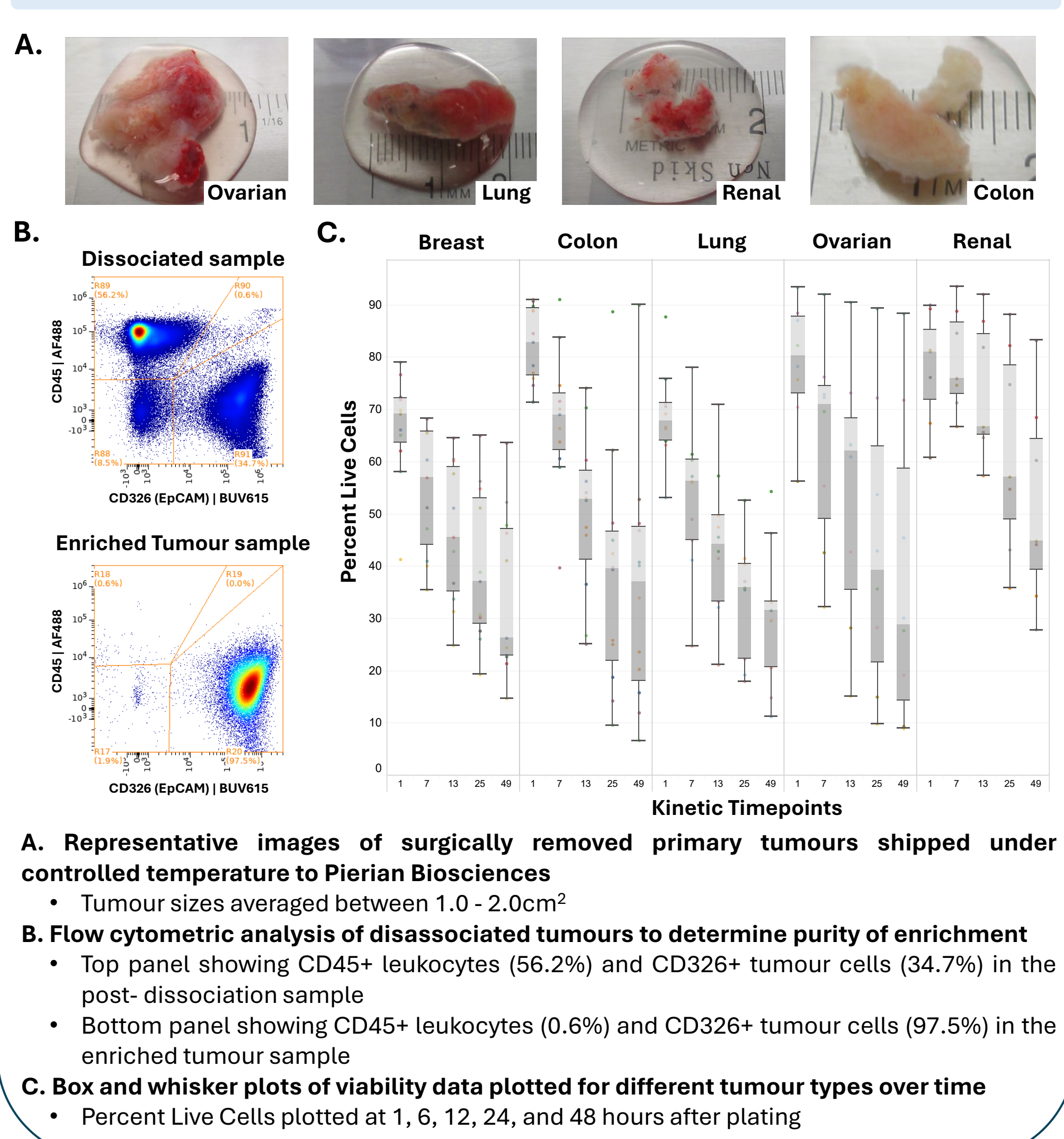

RESULTS

Figure 6. Drug-induced apoptosis is readily observed in tumour cells isolated from patient specimens treated with chemotherapeutic agents as single drugs or in combination

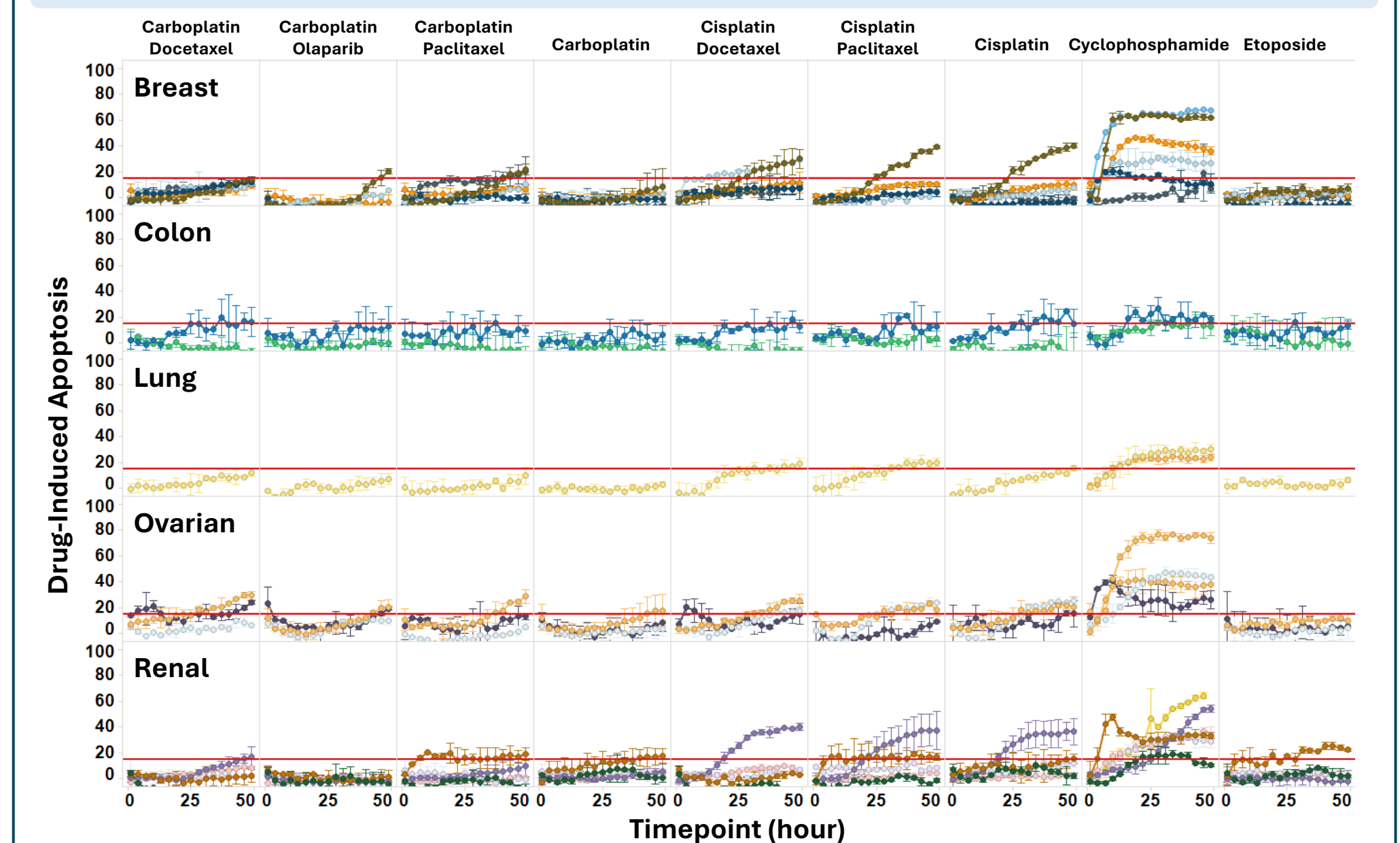

Enriched tumour cells treated with chemotherapeutic agents and fluorescent probes for real-time imaging of cell viability and drug-induced apoptosis for different tumour types

- Cisplatin, Carboplatin, Docetaxel, Paclitaxel, Olaparib, Etoposide and Cyclophosphamide
- Single drug treatments and combination treatments

Figure 7. ChemoINTEL™ can detect a wide range of cellular response as well as informing on potential synergistic effects of combination therapies


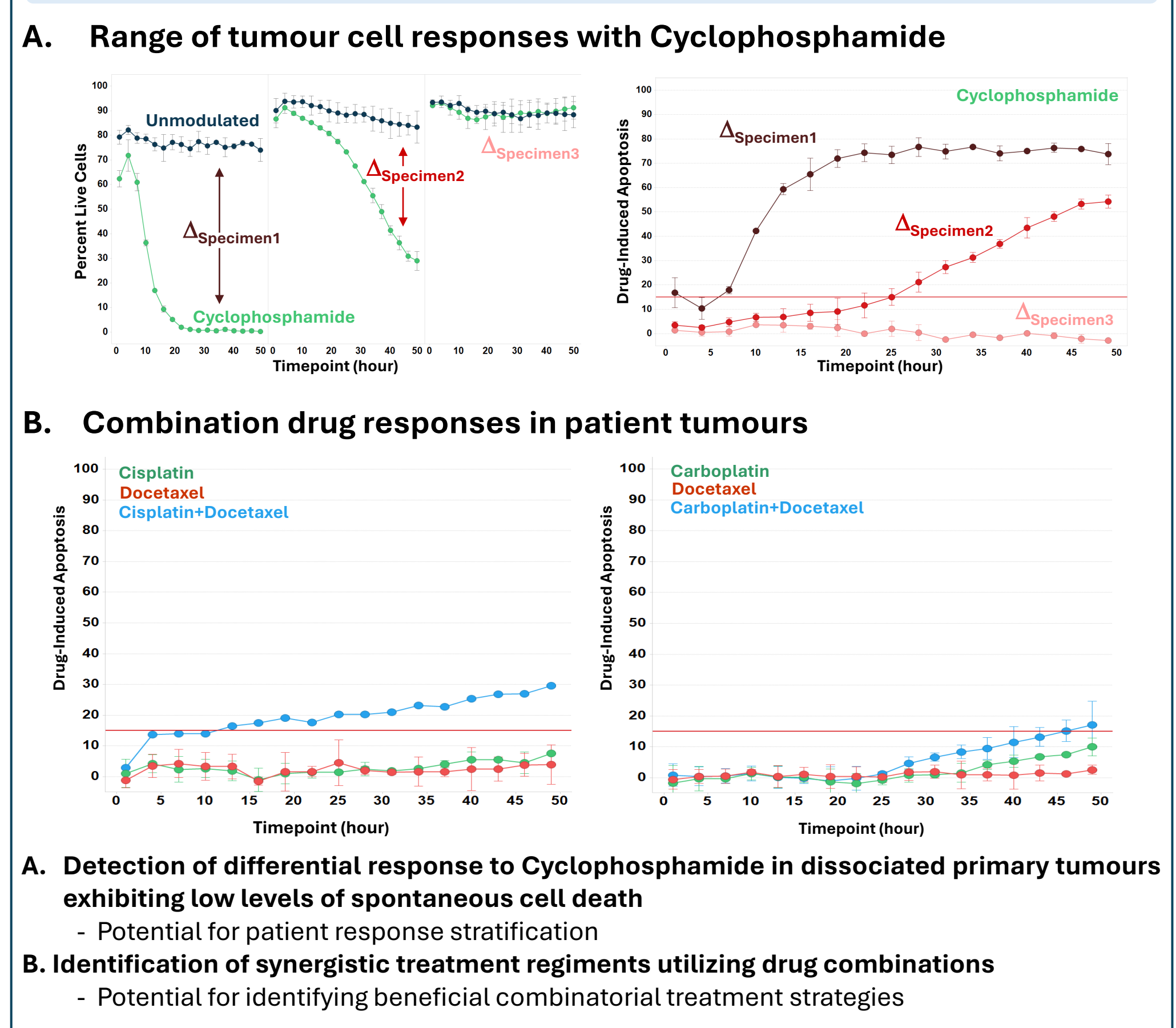


Figure 8. Ongoing algorithm development study

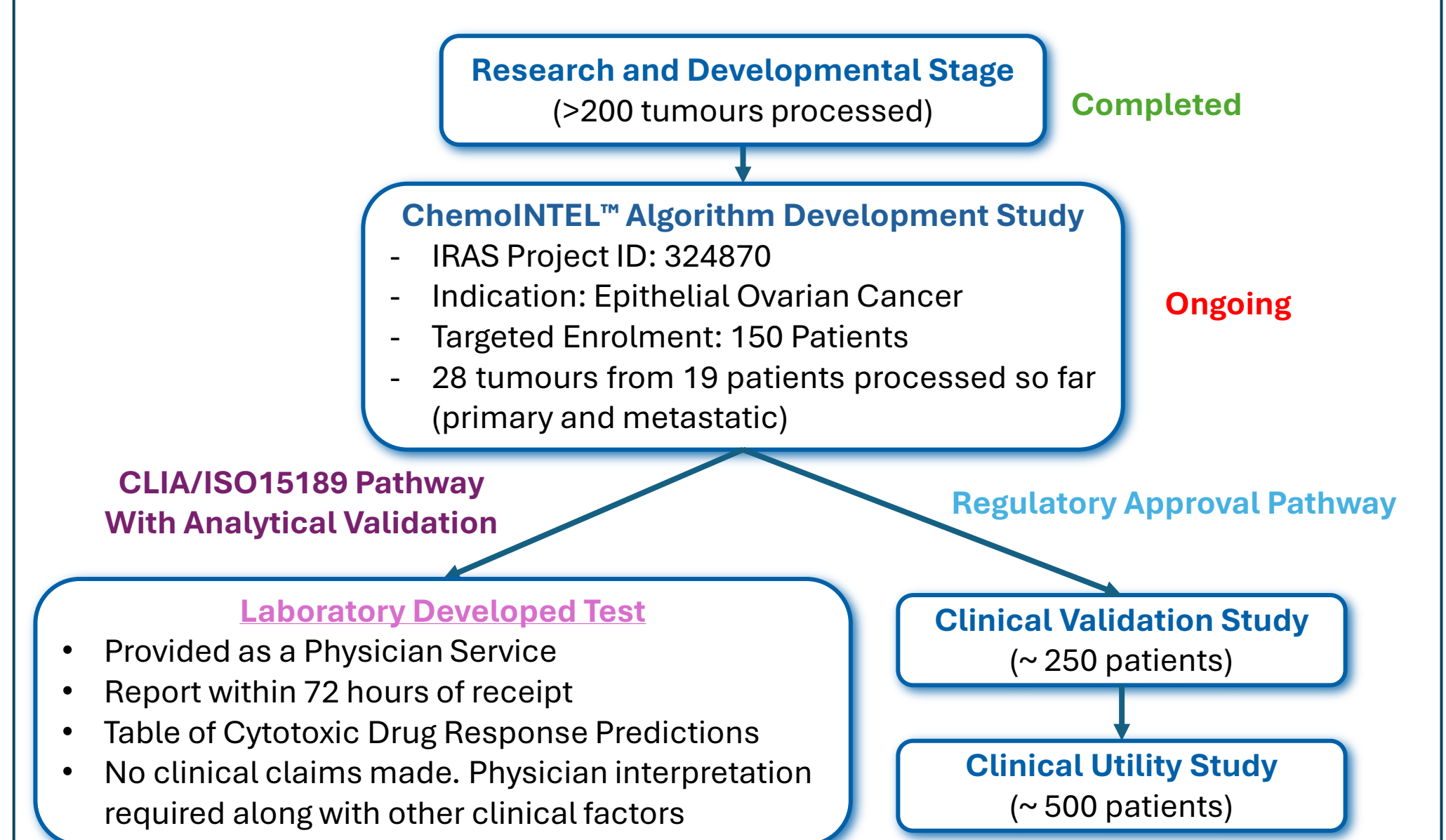

CONCLUSIONS

- Pierian Biosciences has developed a high-throughput, semi-automated, multi-parametric platform intended for use in both drug development and diagnostic applications
- The ChemoINTEL™ platform provides detailed information about effective drug concentrations and kinetic drug profiles
- The downstream software-based HTP analysis accelerates data analysis/extraction and increases the quality of data obtained from the control JURL-MK2 cell line and enriched tumour cells from patient-derived specimens
- The ChemoINTEL™ platform can serve as a drug discovery/evaluation tool (a) in early-stage drug development; (b) for patient stratification in clinical trials; (c) for identifying the most effective treatment regimen in the clinic
- The ChemoINTEL™ Algorithm Development Study is ongoing to model and develop a prediction algorithm for patient tumour cell sensitivity to chemotherapy drugs which will lay the foundation for Clinical Validation and Utility studies.

FOR MORE INFORMATION

Please see poster P-082 for more information about Pierian Biosciences ImmunoINTEL™ flow cytometry platform to support biopharmaceutical assay and drug development

US Headquarters: 479 Sam Ridley Pkwy Suite 105 PMB 292, Smyrna, TN 37167

UK Laboratory: 131 Mount Pleasant Liverpool Science Park, Liverpool L3 5TF

Website: www.pierianbio.com

Clinical Partners:

- The Clatterbridge Cancer Centre
- Liverpool Women's Hospital



PIERIAN BIOSCIENCES

Copies of this poster obtained through Quick Response (QR) Code are for personal use only and may not be reproduced without permission from BGS and the author of this poster.